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# LUMEX®

A Graham-Field Brand

- Soft, contoured foam handle provides a comfortable, secure handgrip
- Easy height adjustment from 31" – 40" in 1" increments with locking mechanism for added security
- Attractive, designer finish
- 250 lb maximum weight capacity
- Lifetime Limited Warranty

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at [www.grahamfield.com](http://www.grahamfield.com)

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© 2012, GF Health Products, Inc.  
2935 Northeast Parkway, Atlanta, Georgia 30360  
Made in Taiwan [www.grahamfield.com](http://www.grahamfield.com)

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### User Instructions and Warranty Information

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## Instructions for Use – Please Read all Instructions Before Use.

This cane is designed for users with recommended heights ranging from 4' 10" to 6' 4". Please consult a healthcare professional for the proper height adjustment and usage.

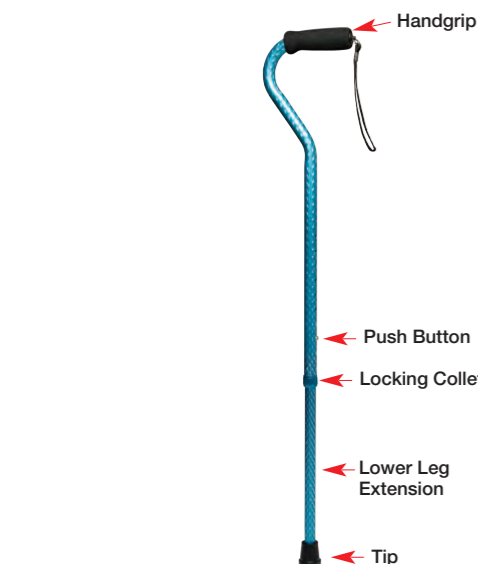
### Adjusting Cane Height

1. Loosen locking collet by twisting counter-clockwise.
2. Adjust cane height: Depress push button and slide lower section upward or downward to desired height.

*NOTE: An audible "click" will be heard when push button securely protrudes through the adjustment hole.*

3. Secure locking collet by turning clockwise until tight.

5950SB	Blue	6/cs
5950SBK	Black	6/cs
5950SR	Red	6/cs
5950SS	Silver	6/cs

## Safety Guidelines – Please Read Before Use

1. DO NOT install or use this equipment without first reading and understanding these instructions. This product should not be used without proper instruction from a healthcare professional.
2. Each individual should always consult with their physician or therapist to determine proper adjustment and usage.
3. This product has a maximum user weight limitation of 250 lb. The canes are NOT designed to support the total weight of individuals.
4. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
5. Before use, ensure cane tip is properly seated and straight. Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.
6. Single push button height adjustment MUST be fully engaged and protruding through height adjustment hole on cane to ensure a positive lock. ALWAYS test to see that cane handle is properly and securely locked in place BEFORE using.
7. Make sure locking collet is securely tightened at all times.
8. Always use caution when using this product on wet or slippery surfaces.
9. If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.
10. Before use, inspect cane and ensure that it is locked securely in position. Inspect weekly.
11. If any component is not in proper working order or appears worn, contact your dealer for repair and/or appropriate replacement parts.

### Maintenance

- Wipe cane weekly with a damp cloth.
- Grips may be cleaned with mild soap or detergent.
- DO NOT use organic solvents, such as acetone, which may damage grip.
- Never use abrasive cleaners or scouring pads on any part of the cane.
- Check locking collet weekly for tightness.
- Replace cane tip as needed.
- To guarantee the best service from your cane, always specify Lumex replacement parts, available from your Graham-Field Distributor.

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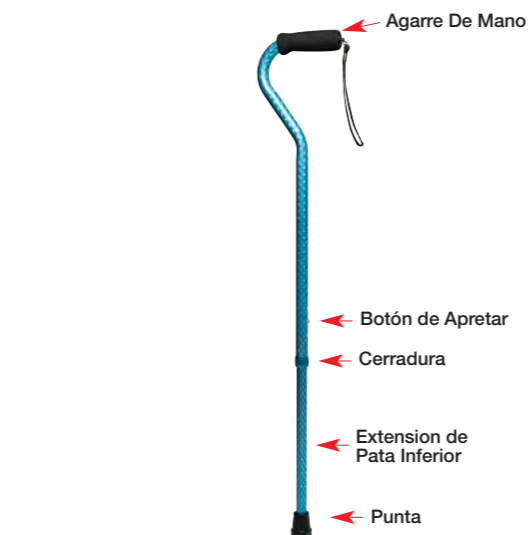
## Lifetime Limited Warranty

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products. We are dedicated to providing products that will give our customers years of dependable service.

GF Health Products, Inc. hereby warrants specific parts of the Lumex models listed on the cover of this hangtag for the lifetime of the product against defective workmanship and/or material defects when used for the purpose intended, under normal conditions, and provided the product receives proper care.

Parts covered under the lifetime warranty include: frame, welding, screws, and frame locking components. Specific items "excluded" from this warranty of any kind are: "ALL" other parts not specifically listed above. The warranty does not extend to non-durable components, such as tips, which are subject to normal wear and require periodic replacement. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to the manufacturer.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this Lumex product or this warranty, please contact an authorized GF Health Products, Inc. Distributor.

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