## Garantía Limitada

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## Instructions for Use – Please Read All Instructions Before Use.

### Setup

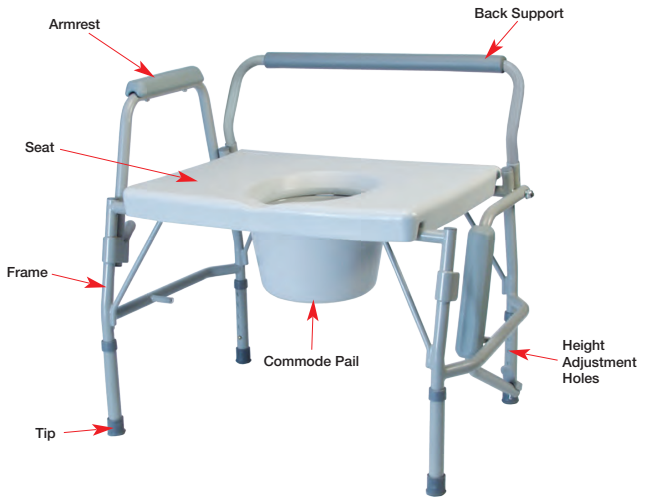
- Remove frame, commode platform and back support from box. Unwrap all packaging materials.
- Commode platform is already attached to frame, as shown below; ensure that all sides of platform are secured to frame.
- The commode comes in its lowest position. Adjusting to desired height is recommended before use.

### Arm Adjustment

- Drop Armrest:** to drop the armrest, push the release lever inwards to release the armrest. This releases the armrest allowing for the transfer on and off the commode.
- Raise Armrest:** to raise the armrest, lift the armrest to the upright position, ensuring that the push button protrudes fully from locking hole.

### Adjusting Leg Height

- Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.  
**NOTE:** An audible “click” will be heard.
- Repeat for other 3 legs.  
**CAUTION:** Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

# LUMEX®

### Features

- Arms drop independently below seat level for lateral transfers (6438A only)
- Arms automatically lock in raised position
- Extra wide platform seat is comfortable for long periods of sitting
- Welded steel construction and extra wide base provide added stability
- Cushioned armrests enhance user comfort
- Comes complete with commode pail, cover and splash guard
- Removable back bar facilitates use as a bedside commode, raised toilet seat or toilet safety frame
- 6438A – 600lb maximum weight capacity EVENLY DISTRIBUTED 7446A-2 – 400lb maximum weight capacity EVENLY DISTRIBUTED
- Designed to meet the requirements of HCPCS code: E0168

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at [www.grahamfield.com](http://www.grahamfield.com)

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© 2014, GF Health Products, Inc.,  
2935 Northeast Parkway, Atlanta, Georgia 30360  
Made in China. [www.grahamfield.com](http://www.grahamfield.com)

GF0900148RevB14

## Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

## WARNING

**WARNING:** DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

**WARNING:** Users with limited physical capabilities should be supervised or assisted when using the commode.

**WARNING:** Item 6438A has a maximum weight capacity of 600 lb. EVENLY DISTRIBUTED  
Item 7446A-2 has a maximum weight capacity of 400 lb. EVENLY DISTRIBUTED

**WARNING:** GF Health Products Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.

**WARNING:** Leg extensions with rubber tips MUST be in contact with the floor at all times.

**WARNING:** Ensure that the push buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.

**WARNING:** Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.

**WARNING:** Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.

**WARNING:** The commode seat MUST be installed before sitting on the commode.

**WARNING:** Notice for California Customers  
**California Proposition 65 WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

**WARNING:** If components are damaged or missing, contact your GF Health Products Inc. distributor immediately.

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### User Instructions and Warranty Information

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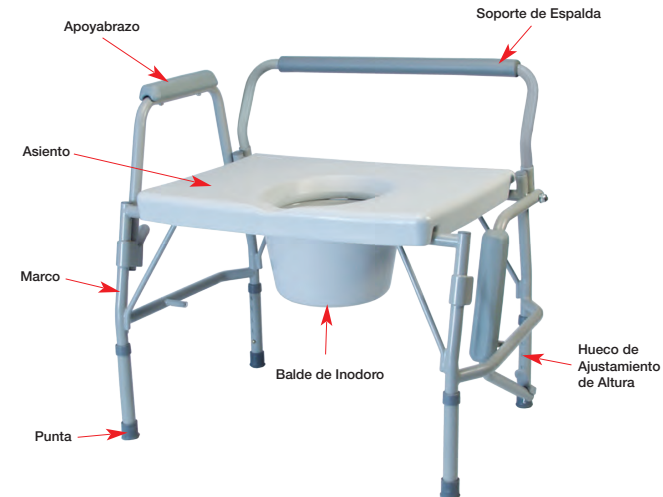
## Limited Warranty

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

**ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER**  
THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

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