7921R / 7921KD / 7931R / 7931KD: SERIES  
BATH SEAT

## ASSEMBLY & OPERATION INSTRUCTIONS

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

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*Info: The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com).*

### **IMPORTANT SAFETY INFORMATION: PLEASE READ**

- ⚠ WARNING: Important! Read and understand these instructions before assembling or using the Bath Seat. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Bath Seat is not properly assembled and adjusted, personal injury and damage to the Bath Seat could result.**
- ⚠ WARNING: If components are damaged or missing, contact your Graham-Field distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Bath Seat.**
- ⚠ WARNING: Notice for California Customers- California Proposition 65  
WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

### **INTENDED USE**

The Bath Seat is intended for use by individuals who require seating support while bathing or showering. It is designed to be placed fully inside the bathtub or shower.


### **Contraindications**

- ⚠ WARNING: This product has a maximum weight capacity of 350 lb, EVENLY DISTRIBUTED.**
- ⚠ WARNING: DO NOT use this product as a transfer bench or climbing device.**
- ⚠ WARNING: Use this product only with tub floors wider than 18 3/4".**

### **ASSEMBLY**

Remove the Bath Seat from the box and place it on a flat, dry surface.

## **Attaching the Legs**

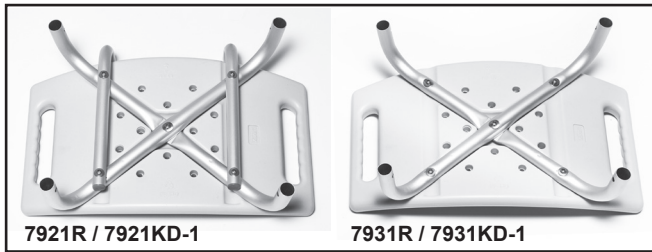**fig. 1**

**fig. 2**

**fig. 3**

1. Locate the seat frame (see fig. 1); place it upside down on a flat, dry surface.
2. Locate the legs (see fig. 2).
3. Attach the legs to the seat frame (see fig. 3). Depress the locking button at the top of each leg and slide it into the seat frame until the button protrudes from the hole.
4. Repeat step 3 for each leg.

**⚠ WARNING: Ensure legs are securely installed and all hardware is tight before proceeding.**

## **Attaching the Backrest: (Model 7921R and 7921KD Series Only)**

**fig. 4**

**fig. 5**

1. Locate the backrest (see fig. 4).
2. Attach the backrest to the seat frame (see fig. 5). Depress the locking buttons and slide the backrest onto the seat frame until the buttons protrude from the holes.

**⚠ WARNING: Ensure backrest is securely installed and all hardware is tight before proceeding.**

## **All Models**

**⚠ WARNING: Ensure the Bath Seat is assembled as described. If the Bath Seat is not properly assembled and installed, personal injury and damage to the Bath Seat could result.**

## **OPERATION**

See assembled Bath Seat at right. Contact your GF distributor immediately if you see any issue with fit, form, function, or safety. For more information on GF Health Products, Inc. parts and services, please contact your GF distributor or [www.grahamfield.com](http://www.grahamfield.com).

### **Adjusting Leg Height**

Push in the adjustment button and move to the desired height position until it clicks into place. Ensure legs are locked at the same height and all leg adjustment buttons protrude fully through adjustment holes.

**⚠ WARNING: Ensure all Bath Seat components are secure before each use.**

### **Center of Balance During Use**

The point where the Bath Seat will tip forward, rearward or sideward is determined by its center of balance and stability, and the center of balance and stability is affected by the user's body position and weight distribution. If you fail to heed the following warnings, falling or tipping may occur and could cause severe personal injury.

**⚠ WARNING: To avoid falling or tipping:**

**Have someone help you until you know the balance points of your Bath Seat and how to avoid tipping.**

**Do not put downward weight on one side or corner of the Bath Seat.**

**Do not put downward weight on the backrest.**

**Do not reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach.**

**Never try to reach an object if you must move forward in your seat and your buttocks do not stay in full contact with the seat.**

**Never lean backward to reach an object.**

## **MAINTENANCE**

**⚠ WARNING: Check the Bath Seat weekly for signs of damage or cracking. If damage or cracking of any component of the Bath Seat is found, DO NOT use the Bath Seat. Contact your Graham-Field distributor.**

Inspect the Bath Seat weekly and ensure all components are secure.

To clean the Bath Seat, wash it with soap and water or a mild detergent, then rinse. Ensure the Bath Seat is rinsed well before use.

**fig. 6**

## **LIMITED WARRANTY**

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for three (3) years; seat/back, extension legs, and arms are warranted for six (6) months; and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

## **OBTAINING WARRANTY SERVICE**

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative.

## **EXCLUSIONS**

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not properly installed, used, or cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

## **ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER**

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© 2005 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China

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