

Info: The most current version of these instructions can be found online at www.grahamfield.com.

Installation tools are provided (open-end M10 x M11 wrench and M4 hex key)

- ⚠ **WARNING:** Important! Read and understand these instructions before assembling or using the Grafco 8146 Utility Cart. If you do not understand any part of these instructions, contact your GF authorized distributor for direction in the use of this product. If the Utility Cart is not properly assembled, personal injury and/or damage to the Utility Cart could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your GF authorized distributor immediately. DO NOT use substitute parts. Use only Grafco replacement parts.
- ⚠ **WARNING:** The 8146 Utility Cart has a maximum load capacity of 400 lb.
- ⚠ **WARNING:** Notice for California Customers- California Proposition 65 **WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.


ASSEMBLY (REFER TO PICTURES AT RIGHT)

- ⚠ **WARNING:** Assembly, repair and maintenance must be performed by qualified personnel only.

The 8146 Utility Cart ships unassembled, as shown at right. Included components are listed below. Items 3, 4, 5, 6 and 7 are packaged in a bagged hardware kit.

8146 Utility Cart Components

Item	Qty	Description	Item	Qty	Description
1	3	Shelf	4	12	Lock Nut M6
2	2	Leg (with Casters)	5	12	Lock Washer M6
3	12	Button Socket Head Cap Screw M6 x 35	6	1	Open-End Wrench
			7	1	M4 Hex Key

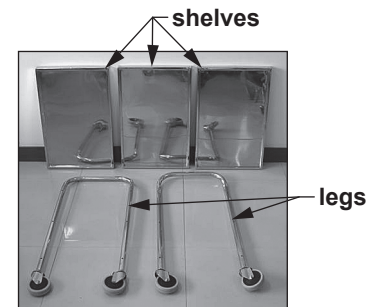

**8146 Utility Cart
unassembled**

1. Carefully remove packing from components.
2. Place shelves between legs, ensuring that handles at tops of legs face outward, and that shelves are positioned as shown at lower right.
3. Align leg holes with shelf holes as shown.
4. Install lock washers on screws. From outside, insert screws through leg holes and shelf holes. Install lock nuts on screws.
5. Ensure the legs are attached to the shelves in an upright perpendicular position and the shelves are level as shown.
6. Use the provided tools to tighten the screws securely.

OPERATION

- ▲ **NOTICE:** Ensure all components are secure before each use.

The Utility Cart may be pushed from either end.



**8146 Utility Cart
assembled**

MAINTENANCE, CLEANING AND STORAGE

▲ **NOTICE:** Replace cracked, damaged, corroded or damaged components immediately.

▲ **NOTICE:** Do not use acid, aggressive disinfectant or corrosive cleaner to clean this product.

▲ **NOTICE:** Ensure Utility Cart is completely dry before packing or storing, and store only in dry conditions.

To clean Utility Cart, wipe with a clean, soft paper or cloth. To disinfect Utility Cart, wipe with a clean, soft, paper or cloth dampened with a general purpose, noncorrosive cleaner, rinse with clear water, and dry with a clean, soft paper or cloth.

WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for one (1) year. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 770-368-4700, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. All illustrations, specifications, packaging and warranties contained in this literature are based on the latest product information available at the time of printing. The most current product information, including the most current version of these instructions, can be found online at www.grahamfield.com. Graham-Field and Grafo are registered trademarks of GF Health Products, Inc.

© 2008 GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, GA 30360, tel 770-368-4700, fax 770-368-2386 • Made in India