Bath Stool

Taburete de baño

Items 2060A, 2060R

SAFETY GUIDELINES - PLEASE READ BEFORE USE

- **DO NOT** install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Item has a maximum weight capacity of 300 lb - **EVENLY DISTRIBUTED**.
- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- **ALWAYS** inspect the bath seat to ensure that it is properly positioned and stable before using. **DO NOT** use if bath seat is wobbly or unstable.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. **DO NOT** use substitute parts.
- Check leg tips for rips, tears, cracks or wear. If any of these conditions exist, replace leg tips **IMMEDIATELY**.
- **DO NOT** use this as a transfer bench or climbing device.
- To avoid falling or tipping: Have someone help you until you know the balance points of your Bath Seat and how to avoid tipping. Do not put downward weight on one side or corner of the Bath Seat. Do not put downward weight on the backrest. Do not reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach. Never try to reach an object if you must move forward in your seat and your buttocks do not stay in full contact with the seat. Never lean backward to reach an object.
- All four leg tips **MUST** be in contact with shower/tub floor at all times.
- Ensure that the four chair legs are adjusted to the same height and that the push buttons fully protrude through their respective adjustment holes **BEFORE** use.
- Only use GF Health Products, Inc. accessories with this product.
- This product should only be used with tub floors wider than 18 $\frac{3}{4}$ inches.
- Users with limited physical capabilities must be supervised or assisted when using the bath seat.
- Make certain **ALL** screws and nuts are tight at all times.
- **DO NOT** submerge product in water, otherwise damage may result.

MAINTENANCE

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

ASSEMBLY

1. Lay bath seat upside down with upper legs pointing up on a flat, dry surface.
2. Attach lower leg by pushing in the locking button and sliding the lower leg over the upper leg until it clicks into place. Repeat for other three legs. Ensure all legs are locked before proceeding.

NOTE: *Ensure seat support tube buttons protrude fully through channel guide holes.*

ADJUSTING LEG HEIGHT

Legs are already installed on the bath seat.

To adjust leg height:

1. Press push button and slide leg extension up or down to desired adjustment hole.
2. Ensure push button fully protrudes through the leg adjustment holes of the leg extension.

NOTE: *An audible “click” will be heard.*

3. Repeat STEPS 1 and 2 for the other three legs.

NOTE: *Ensure all four legs are adjusted to the same height and that all height adjustment buttons protrude fully through the adjustment holes before use.*

LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling **(678) 291-3207**, sending a fax request to **(770) 368-2386** or by e-mailing a request to **cs@grahamfield.com**. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: **1)** Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; **2)** Products not installed, used, or properly cleaned and maintained; and **3)** Labor or shipping charges. Other exclusions apply. Additional limitations relative to this warranty may be found on www.grahamfield.com Terms and Conditions page. State laws governing the warranty of this product may provide more expanded rights than those set forth in this warranty.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

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Punta

Features

- Anodized aluminum frame is lightweight, durable and rust-resistant
- Seat height adjusts in 1" increments for various user heights
- Tool-free assembly
- Maximum Weight Capacity 300 lb EVENLY DISTRIBUTED

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2017, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com