LUMEX[®]
by graham-field

***LX9000 Gaitster Forearm Rollator
User Manual***

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LX_LX9000-INS-LAB-RevC21

**Read this manual before assembling
or using the Rollator.
Save this manual for future reference.**

CONTENTS

INTRODUCTION.....	2
PRODUCT DESCRIPTION.....	2
INTENDED USE.....	3
SAFETY GUIDELINES—PLEASE READ BEFORE USE.....	3
WARNING / CAUTION SUMMARY.....	3
RAMPS AND INCLINES.....	4
CURBS AND STEPS.....	4
UNPACKING.....	5
ASSEMBLY.....	6
LEG FRAME ASSEMBLY.....	6
FRONT BRAKE CABLE CLIP INSTALLATION.....	6
HANDLE ASSEMBLY.....	6
CANE / CRUTCH HOLDER INSTALLATION.....	6
STORAGE POUCH INSTALLATION.....	7
ADJUSTMENT.....	7
HANDLE HEIGHT ADJUSTMENT.....	7
HANDGRIP / HAND BRAKE POSITION ADJUSTMENT.....	7
OPERATION.....	8
FOREARM ROLLATOR SETUP.....	8
WALKING WITH YOUR FOREARM ROLLATOR.....	8
STOPPING WITH YOUR FOREARM ROLLATOR.....	8
TO SIT IN FOREARM ROLLATOR.....	8
CLEANING AND MAINTENANCE.....	9
TRANSPORT AND STORAGE.....	9
SPECIFICATIONS.....	10
LIMITED WARRANTY.....	11

INTRODUCTION

This manual contains assembly, operation and maintenance instructions for your rollator. Read the entire manual carefully before using your rollator, and refer to it during use if you have questions.

PRODUCT DESCRIPTION

INTENDED USE

The intended use of the LX9000 Gaitster Forearm Rollator is as a rollator, a walking aid with wheels, manipulated by both arms.

⚠ WARNING: This product has a maximum weight capacity of 300 lb, EVENLY DISTRIBUTED.

SAFETY GUIDELINES—PLEASE READ BEFORE USE

The safety statements presented in this chapter refer to the basic safety information that the rollator user shall pay attention to and abide by. There are additional safety statements in other chapters or sections, which may be the same as or similar to the following, or specific to the operations. Please note the following special statements, used throughout this manual, and their significance:

⚠ WARNING: Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in death or serious personal injury.

⚠ CAUTION: Indicates a potential hazard or unsafe practice that, if not avoided, could result in minor or moderate personal injury.

▲ NOTICE: Indicates a potential hazard or unsafe practice that, if not avoided, could result in product or property damage.

Info: Provides application recommendations or other useful information to ensure that you get the most from your product.

WARNING / CAUTION SUMMARY

⚠ WARNING: Important! Read and understand these instructions before assembling or using the rollator. Ensure that the rollator is assembled as described in the following pages before use. If the rollator is not properly assembled and adjusted, personal injury and/or damage to the rollator could result.

⚠ WARNING: DO NOT use this product without proper instruction from a healthcare professional.

⚠ WARNING: GF Health Products, Inc. ("Graham-Field") assumes no responsibility for any damage or injury caused by improper assembly or use of this product.

⚠ WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.

⚠ WARNING: Inspect the rollator weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your authorized GF distributor for repair and/or appropriate replacement parts.

⚠ WARNING: DO NOT operate the rollator on streets or roadways.

⚠ WARNING: DO NOT operate the rollator on hilly or rough terrain, sand, wet, slippery or icy surfaces, or surfaces with impaired traction. Ensure that pathway is clear of all obstacles.

⚠ WARNING: The rollator does not offer seating or occupant restraint equivalent to the seat provided in a motor vehicle! To increase your safety while traveling in a motor vehicle, always transfer to the vehicle seat and use the restraint provided by the vehicle manufacturer.

⚠ WARNING: DO NOT tie down or attach anything to the wheels. This could cause tipping and possibly result in injury or damage to the rollator.

⚠ WARNING: Ensure that the rollator is on a stable, level surface and engage handbrakes before and during transfer or using an elevator.

- ⚠ WARNING: Unauthorized modification or the use of non-Lumex® replacement parts could change the structure of the rollator, void the warranty, and create a hazardous condition resulting in serious personal injury.**
- ⚠ WARNING: DO NOT use the rollator on stairs or escalators.**
- ⚠ WARNING: Cancer and Reproductive Harm - www.p65warnings.ca.gov.**
- ⚠ WARNING: GF Health Products, Inc. specifically disclaims responsibility for any personal injury or property damage which may occur during any use which does not comply with federal, state, or local laws or ordinances.**

RAMPS AND INCLINES

- ⚠ WARNING: DO NOT leave the rollator on an incline.**
- ⚠ WARNING: DO NOT turn the rollator while going downhill, as the rollator could become unstable or tip over.**
- ⚠ WARNING: DO NOT attempt any incline or decline greater than six degrees (10% grade, or one foot of rise or fall per ten feet of ramp length).**

Ramps and inclines should only be negotiated with the assistance of an attendant. To attendant: Practice with a healthcare professional first before attempting ramps or inclines. Always inspect the ramp for hazards such as holes, slippery or uneven surfaces, etc. before starting up or down. If you can not see the entire ramp, ask someone to inspect it for you.

CURBS AND STEPS

- ⚠ WARNING: Never attempt to negotiate curbs or steps with your rollator without the assistance of an attendant.**
- ⚠ WARNING: Never attempt to negotiate stairs or escalators with your rollator under any circumstances.**

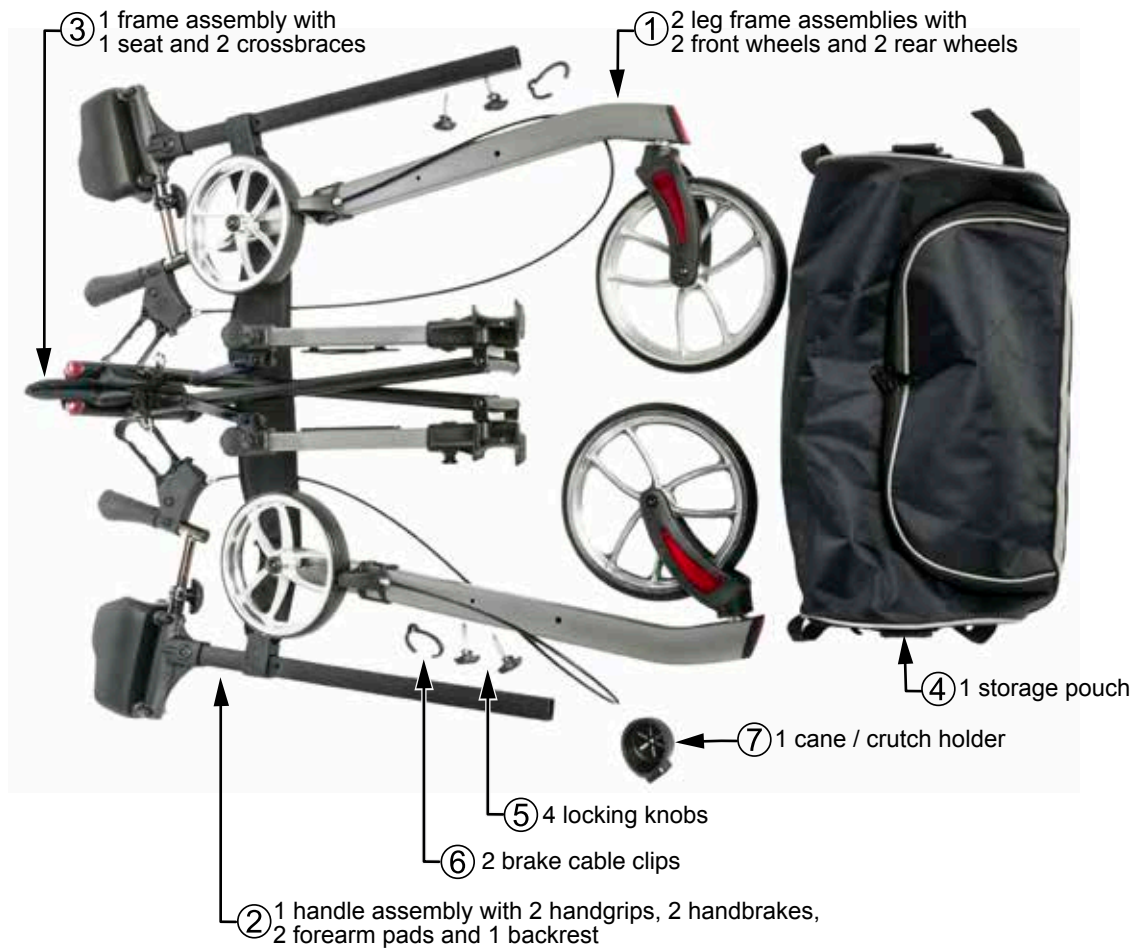
Curbs, steps and stairs are dangerous obstacles that confront the rollator user. The rollator is equipped with a curb stepper that can assist you when tilting the rollator to get over taller obstacles like curbs. When you encounter steps and stairs, find a way around, or use the ramps now available in most locations. If you encounter stairs or escalators and there is no ramp available, avoid the stairs or escalators by utilizing the disabled designated elevators now required in most locations. Curbs and steps should only be negotiated with the assistance of an attendant.

UNPACKING

Unless the rollator is to be used immediately, retain box and packaging materials for storage until use is required.

1. Check for obvious damage to the carton or its contents. If damage is evident, please notify the carrier and your GF authorized distributor.
2. Remove all loose packing from the box.
3. Carefully remove all the components from the carton.

Each box should contain the following components (see picture below):

Item	Qty	Description
1	2	leg frame assemblies with 2 front wheels and 2 rear wheels
2	1	handle assembly with 2 handgrips, 2 handbrakes, 2 forearm pads and 1 backrest
3	1	frame assembly with 1 seat and 2 crossbraces
4	1	storage pouch
5	4	locking knobs
6	2	brake cable clips
7	1	cane / crutch holder

Info: Items 1 and 2 are connected by brake cables.

ASSEMBLY

LEG FRAME ASSEMBLY

1. Position the rollator frame assembly and leg frame assemblies on their sides as shown at near right.
2. Install the leg frame assemblies on inside of rollator frame assembly as shown at right.

Info: for correct assembly, ensure wheel / fork reflectors and rear wheel braking system are on outside of frame.

3. Tighten both screws on outside of leg frame assemblies as shown above.

⚠ WARNING: Ensure the leg frame assemblies are securely installed before proceeding.

FRONT BRAKE CABLE CLIP INSTALLATION

Ensure there is no interference with the brake cable, and fasten the brake cable clip on outside of leg frames, in front of frame junction, as shown above. This will help prevent the brake cable from rubbing or being caught on anything.

HANDLE ASSEMBLY

⚠ WARNING: During assembly, do not place any fingers or parts of your hand under the seat support bars.

1. Position the rollator in upright position.
2. Push down on both sides of the seat to securely lock the seat support bars into place as shown at right.
3. Install both handles in both frame tubes as shown at right.
4. Move handles to desired height — note numbers on handle shafts shown at right to match handle height.
Ensure both handles are adjusted to the same height.

5. Turn locking knobs clockwise to lock handles in place inside frame.

⚠ WARNING: Ensure locking knobs are securely tightened before proceeding.

CANE / CRUTCH HOLDER INSTALLATION

Push the cane / crutch holder into the frame connector until it snaps into place, as shown at right.

⚠ WARNING: Ensure the cane / crutch holder is securely installed before use.

STORAGE POUCH INSTALLATION

1. Install the storage pouch on the frame's wire hangers, shown at near right just below front of seat.
2. Secure hook and loop straps around both legs, as shown at middle right.
3. Secure hook and loop straps around front of frame just behind front wheel, as shown at upper far right, where hook and loop straps are shown fastened over front brake cable clip.

The storage pouch, shown correctly installed at right, can be conveniently removed to place items and then reattached, no tools needed.

⚠ WARNING: Ensure the storage pouch is securely installed before use.

ADJUSTMENT

HANDLE HEIGHT ADJUSTMENT

Info: Consult your healthcare professional, distributor or caregiver to assist you in finding the best handle height for you.

1. Turn both locking knobs, shown at right, counterclockwise to release handles inside frame.
2. Move handles to desired height — note numbers on handle shafts shown at right to match handle height.
Ensure both handles are adjusted to the same height.
3. Turn locking knobs clockwise to lock handles in place inside frame.

⚠ WARNING: Ensure locking knobs are securely tightened before use.

HANDGRIP / HAND BRAKE POSITION ADJUSTMENT

Info: The handgrips can be adjusted to an angle and depth most comfortable for the user.

Refer to picture at right.

1. Loosen both locking handles.
2. Move handgrip / handbrake assembly tubing to desired angle and depth.
Ensure both handgrips are adjusted to the same depth.
3. Tighten both locking handles to lock handgrip / handbrake assemblies in place.

⚠ WARNING: Ensure locking handles are securely tightened before use.

OPERATION

⚠ **WARNING:** Read the safety section of this manual before use.

⚠ **WARNING:** Before using the rollator, ensure the wheels are securely attached to the frame; the seat is fully in its locked position; the backrest is securely installed; the forearm pad / handgrip / handbrake is securely installed, locked, and adjusted to the appropriate user depth; and the handles are securely installed, locked, and adjusted to the appropriate user height.

Before proceeding, ensure all instructions in the previous section, **ASSEMBLY**, have been followed, and that you have read the **SAFETY** section of this manual.

FOREARM ROLLATOR SETUP

The photo at right shows the fully assembled rollator. To keep the backrest out of the way during movement, position it *away* from the handles as shown at right.

WALKING WITH YOUR FOREARM ROLLATOR

⚠ **WARNING:** Do not use resting handles (shown above) to move or push the rollator. The resting handles may be pivoted upward out of the way while walking.

1. Place your forearms on the forearm pads in a comfortable, stable position, where you can easily reach the handgrips and handbrakes.
2. Move the rollator slowly forward.
3. Select a pace that is comfortable for you.

STOPPING WITH YOUR FOREARM ROLLATOR

The rollator has loop locking handbrakes. To slow down, squeeze the handbrakes in toward the handgrips. To hold the rollator in a stationary position, push the handbrakes away from the handles, until the handbrakes lock into place; the rear wheels will then be locked into parked position. To release, pull the handbrakes toward the handgrips until the rear wheels move freely.

TO SIT IN FOREARM ROLLATOR

1. Push the handbrakes away from the handgrips until the rear wheels lock into parked position.
2. Test the wheels before sitting down to ensure the rollator will not roll.
3. Before sitting, touch the back of your legs to the edge of the seat to ensure proper positioning of the rollator.
4. Hold onto rollator handles if necessary to help you lower your body onto the seat.
5. While seated, you may rest your arms on resting handles (shown above).

⚠ **WARNING:** Distribute your body weight evenly on the rollator seat.

⚠ **WARNING:** DO NOT lean on one side of the rollator.

⚠ **WARNING:** Ensure that handbrakes stop rollator and lock in place appropriately before occupying or operating rollator. If the handbrakes are not operating correctly, see your GF authorized distributor immediately for service.

CLEANING AND MAINTENANCE

Use a clean, soft cloth and warm water to clean the rollator frame, handles, and wheels. Rinse and air dry.

▲ **NOTICE:** Do NOT use solvents and other cleaning solutions to clean any part of the rollator.

Check fit and tightness of all nuts, bolts and knobs weekly.

⚠ **WARNING:** Inspect the rollator weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF authorized distributor for repair and/or appropriate replacement parts.

⚠ **WARNING:** The use of non-Lumex replacement parts will void the warranty, and could create a hazardous condition resulting in serious personal injury.

TRANSPORT AND STORAGE

The rollator is designed for easy folding and storage.

To Fold for Transport

1. Remove the storage pouch by pulling off the hook and loop straps that secure the pouch to the frame.
2. Pull the strap in the middle of the seat up to bring the crossbraces together.
3. Push the handles together; the rollator will fold easily and be ready for transport as shown at far right.

To Store

1. Remove the storage pouch by pulling off the hook and loop straps that secure the pouch to the frame.
2. Adjust the handle height to the lowest position.
3. Pull the strap in the middle of the seat up to bring the crossbraces together
4. Push the handles together; the rollator will fold easily and be ready for transport as shown at far right.

SPECIFICATIONS

DIMENSION		LX9000
Width Inside Handgrips		15.25" (387 mm)
Overall Width		27.4" (697 mm)
Overall Width Folded Upright (measured from rear wheels)		13.5" (343 mm)
Overall Depth		30.2" (768 mm)
Handle Height	Minimum	40.2" (1021 mm)
	Maximum	46.8" (1190 mm)
Forearm Pad Size		6.9" x 4.6" x 3.3" (175 mm x 118 mm x 85 mm)
Seat to Floor Height		24" (613 mm)
Approximate Height of Intended User		5'5" - 6'5" (165.1 cm - 195.6 cm)
Maximum Weight Capacity EVENLY DISTRIBUTED		300 lb (136 kg)
Product Weight		23 lb (10.43 kg)
Seat	Width	18" (460 mm)
	Depth	11" (279 mm)
Wheel Size	Front	9.25" (235 mm)
	Rear	8" (203)

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. The warranted components and time period are set forth below:

Frame: three (3) years

Wheels, handgrips, handbrakes, forearm pads, and seat: six (6) months

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: tips, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.grahamfield.com

Follow us on   

© 2020, GF Health Products, Inc. All Rights Reserved.

Lumex is a trademark of

GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140
GF Health Products, Inc. is an ISO 13485:2016 / MDSAP Certified Company.

Manufactured for GF Health Products, Inc.
Made in China

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© 2020, GF Health Products, Inc. Todos los derechos reservados.
GF Health Products, Inc. es una empresa certificada ISO 13485:2016 / MDSAP.
Fabricado para GF Health Products, Inc.