# LUMEX®

by graham-field

## **Silver Collection Low Profile Bariatric Quad Cane**

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graham  
field

QTY. 1 EA

## SAFETY GUIDELINES - PLEASE READ BEFORE USE

1. This product should not be used without proper instruction from a healthcare professional.
2. This product has a maximum user weight limitation of 500 lb. - **EVENLY DISTRIBUTED**. The product is **NOT** designed to support the total weight of individuals.
3. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
4. If components are damaged or missing, contact your distributor immediately. **DO NOT** use substitute parts.
5. Before use, ensure tip is properly seated and straight. Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.
6. Always use caution when using this product on wet or slippery surfaces.
7. Inspect product weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
8. Single push button height adjustment **MUST** be fully engaged and protruding through height adjustment hole on cane to ensure a positive lock. **ALWAYS** test to see that cane handle is properly and securely locked in place **BEFORE** using.
9. Make sure locking collet is securely tightened at all times.
10. When using a Quad cane, the cane should be placed on a level walking surface with all four legs in contact with the surface. Use of cane in a fashion other than this can lead to eventual cane damage and/or personal injury.

## GENERAL WARNING

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. **DO NOT** substitute parts.

## GENERAL WARNING (CONT.)

- Before use, ensure that all assembled parts are secure and all hardware is tightened.
- Care should be taken to ensure that all height adjustments are secure.
- **ALWAYS** observe the user weight limitation on the hangtag of this product.

 **WARNING:** Cancer and Reproductive Harm-  
[www.p65warnings.ca.gov](http://www.p65warnings.ca.gov).

## INSTALLATION WARNINGS

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect product weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- **ALWAYS** test to see that the product is properly secured in open position **BEFORE** use and ensure all hardware is securely tightened.

## MAINTENANCE

- Wipe product weekly with a damp cloth.
- Grips may be cleaned with mild soap or detergent. **DO NOT** use organic solvents, such as acetone, which may damage grip.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check locking collet weekly for tightness.
- Replace cane tip as needed.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## INSTRUCTIONS FOR USE

*Please Read all Instructions Before Use.*

This cane is designed for users with recommended heights ranging from 4' 10" to 6' 4". Please consult a healthcare professional for the proper height adjustment and usage.

## HEIGHT ADJUSTMENT

1. Loosen locking collet by twisting counter-clockwise.
2. Adjust cane height: Depress push button and slide lower section upward or downward to desired height..

**NOTE:** *An audible "click" will be heard when push button securely protrudes through the adjustment hole.*

3. Secure locking collet by turning clockwise until tight.

## LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for three (3) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

## OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling **(678) 291-3207**, sending a fax request to **(770) 368-2386** or by e-mailing a request to **cs@grahamfield.com**. Specific directions will be provided by the Customer Service Representative.

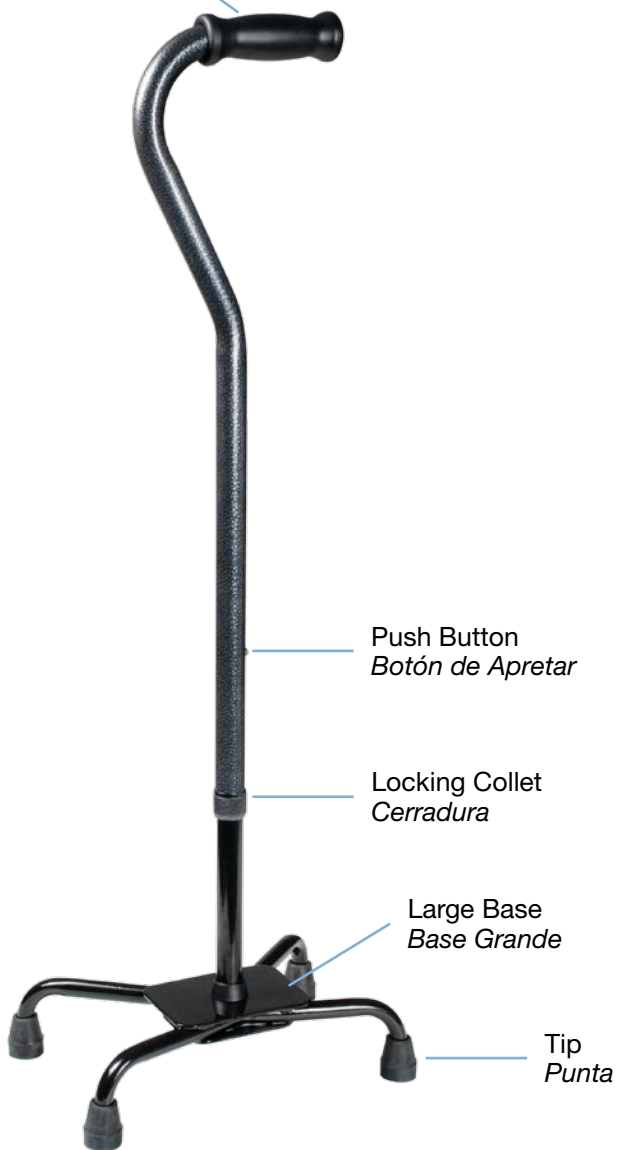
## EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: **1)** Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; **2)** Products not installed, used, or properly cleaned and maintained; and **3)** Labor or shipping charges. Other exclusions apply. Additional limitations relative to this warranty may be found on [www.grahamfield.com](http://www.grahamfield.com) Terms and Conditions page. State laws governing the warranty of this product may provide more expanded rights than those set forth in this warranty.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

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## Features

- Offset handle places user's weight over the base for maximum balance and control
- Stability is enhanced with the low profile, 4-point base
- Easy height adjustment from 29" – 38" in 1" increments with locking mechanism for added security
- Strong and durable rounded base protects user's legs when walking
- 500 lb maximum weight capacity  
- EVENLY DISTRIBUTED

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at [www.grahamfield.com](http://www.grahamfield.com)

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© 2009, GF Health Products, Inc.,  
One Graham-Field Way, Atlanta, Georgia 30340.  
Made in China. [www.grahamfield.com](http://www.grahamfield.com)